NL - DerMel wondverband

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Product

DerMel wondverband

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Product

DerMel wound dressing

8 x 8 cm gauze impregnated with honey for use in wound treatment.

Product description

DerMel wound dressing is a gauze dressing impregnated with pure honey produced under controlled conditions. This controlled production process guarantees that DerMel wound dressing contains no residues of pesticides and is always of the same reproducible high quality. The honey used in DerMel wound dressing is rich in honey enzymes, including glucose oxidase. Thanks to the high sugar concentration, some moisture from the wound exudate mixes with DerMel wound dressing to form a layer that reinforces the antibacterial protection provided by the wound dressing.

How to apply

1. If necessary, clean the wound in the usual way.
2. Open the packaging at the top by tearing the seal open and removing the protective film covering the gauze.
3. Take the gauze dressing out of the packaging and place it over the wound.
4. Secure the gauze in the usual way, e.g. using self-adhesive porous tape.
5. The gauze may also be covered with a secondary bandage, which is then secured in the usual way.
6. Remove the dressing carefully after one to three days (maximum), and replace it with a new DerMel dressing if necessary.

Indications

DerMel wound dressing can be used on various types of wound such as decubitus (pressure sores), ulcers, infected and acute wounds, surgical wounds, oncological wounds resulting from radiation treatment, and 1st and 2nd degree burns.

Contraindication

- Do not use this product if the patient is known to be allergic or hypersensitive to honey. An allergic reaction or hypersensitivity can cause redness, irritation and/or soreness of the skin or wound.
- Please contact your doctor or the manufacturer if any unexpected side effects occur. Side effects can cause the patient discomfort.

Precautions

- The product must not be used if honey is leaking from the inner packaging. Leaking packaging could contaminate the product with bacteria which, in turn, could cause wound infection. Thanks to honey's antibacterial protection, the risk of wound infection is considered very low.
- Only use if the packaging (pouch) is properly sealed. Packaging that is not properly sealed could result in the dressing no longer being sterile and possibly causing wound infection. Thanks to honey's strong antibacterial protection, the risk of wound infection is considered very low.
- Do not apply this product for more than 30 consecutive days. Treatment for more than 30 consecutive days could cause the patient discomfort such as irritation and/or soreness.
- Used dressings must not be reused. Reusing a used dressing could reduce the effectiveness of the honey, contaminate the product, and contaminate or infect the wound.
- Do not resterilise an opened package. Resterilising an opened package could reduce the effectiveness of the honey, contaminate the product, and contaminate or infect the wound.
- Do not use the product after the 'use before' date. This could reduce the effectiveness of the product and cause the patient discomfort.

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Bifactory Health Products B.V. Remmerden 58, 3911 TZ Rhenen (NL)
DOS Medical B.V. Laan van Westenenk 64 7336 AZ Apeldoorn (NL) www.dos-medical.com